


# EasyGrip Flo-41

## PRECISION MIS DELIVERY SYSTEM

## QUICK REFERENCE GUIDE

### 1 Initial & Repeat Product Set-up and Application

Inside the sterile field

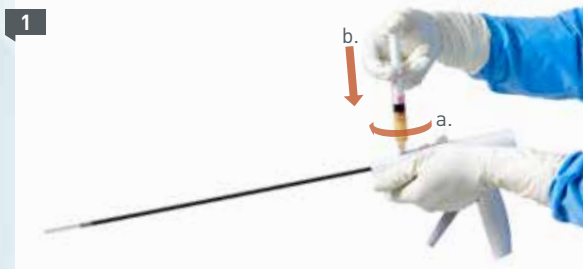

- **a)** Attach the FLOSEAL Matrix syringe to the EASYGRIP FLO-41 fill port. **b)** Fill the reservoir completely with FLOSEAL Matrix. **Syringe may be removed if desired.**




- Prime cannula by repeatedly pulling the trigger (approximately 2-3 pulls) until FLOSEAL Matrix is visible in the clear malleable tip. The EASYGRIP FLO-41 Applicator is now ready to use.



- Pull the trigger of the EASYGRIP FLO-41 to apply the FLOSEAL Matrix, per its instructions. Multiple pulls may be required to apply desired amount of FLOSEAL Matrix. **If additional FLOSEAL Matrix is required during the same surgical procedure, repeat steps 1&3. Priming is not required for repeated loading.**

### 2 Residual Product Application

Inside the sterile field

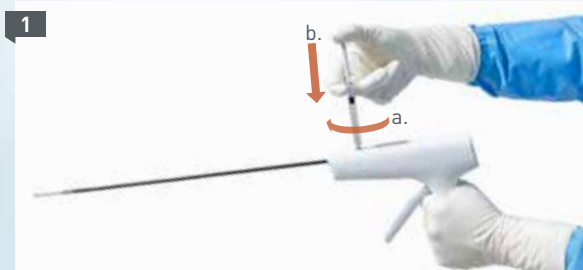

- If application of residual FLOSEAL Matrix is needed, **fill the provided 1.5 mL syringe with non-heparinized saline.** **a)** Attach saline syringe to EASYGRIP FLO-41 and **b)** fill the reservoir with only 1.5 mL of non-heparinized saline.



- Pull the trigger of the EASYGRIP FLO-41 System to apply residual FLOSEAL Matrix at the site of the bleed.



- If, after expelling all residual product from cannula, additional FLOSEAL Matrix application is required, the cannula must be purged of saline. **Remove saline from the cannula and prime the device for additional use.**

## EasyGrip FLO-41 Indication

The EASYGRIP FLO-41 System is indicated for delivering FLOSEAL Hemostatic Matrix to bleeding sites through a 5mm or larger trocar.

## Important Risk Information for EasyGrip FLO-41

Single use only. Do not reuse. Do not resterilize. Reuse or reprocessing of a single use device may lead to contamination and compromised device function or structural integrity.

Do not use the EASYGRIP FLO-41 System if the packaging is damaged or open, or if the applicator is damaged, as this could lead to compromised device function.

Do not use air as a propellant to extrude hemostatic agent from the EASYGRIP FLO-41 System.

Do not use the EASYGRIP FLO-41 System to manipulate or retract organs or tissue.

Do not bend the malleable tip against organs or tissue.

**Rx Only.** For safe and proper use of this device, refer to the full Instructions for Use.

## ORDERING INFORMATION

EasyGrip FLO-41	Box of 6	ADS201865
FLOSEAL Matrix Needle Free, 5mL	Box of 6	1504610
FLOSEAL Matrix Needle Free, 10mL	Box of 6	1504612

For questions or ordering information, please contact your Baxter representative.

**Advancing the art of healing**

Australia  
Baxter Healthcare Pty Ltd  
1 Baxter Drive, Old Toongabbie,  
NSW 2146, Australia  
Ph: +61 1800 229 837  
[www.baxterhealthcare.com.au](http://www.baxterhealthcare.com.au)

New Zealand  
Baxter Healthcare Ltd  
33 Vestey Drive, Mt Wellington,  
Auckland, New Zealand  
Ph: +64 (0) 800 229 837  
[www.baxter.co.nz](http://www.baxter.co.nz)

**Baxter**